

Vector Duo Amalgamator

A-1000

Digital dual speed amalgamator




WARNING! PRIOR TO USE!

Remove the two small wing nut shipping screws prior to the first use of your new Duo Amalgamator!

Operation Instructions:

- 1) Connect the Vector Duo-Amalgamator to a standard 110 volt power outlet.
- 2) Turn the unit on by the main power switch located in the rear of the unit. The “-” symbol is ON. The “o” symbol is OFF. The green mixing time display should light up.
- 3) Select the speed from the two options by pressing the oval speed button.
“L” indicates Low speed (4000 oscillations per sec +/- 5%)
“H” indicates High speed (4500 oscillations per sec +/- 5%)
- 4) Adjust the mixing time from 1 to 99 seconds.
- 5) Place the activated capsule into the clipper by inserting the capsule into the bottom clip first then pulling backwards on the capsule to snap it into place. (see diagram)

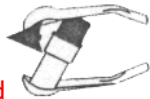

Pull forward

- 6) Close the safety cover completely. **Note the unit will not activate unless the safety cover is completely closed!**
- 7) Start the mixing process by pressing the orange button. To stop the mixing process, press the orange button again.
Note: At the end of the mixing process the unit will remain programmed to the last mixing time even if the unit is switched off or disconnected.
- 8) The machine will stop automatically at the end of the programmed mixing time.

Packing list: Machine, Power Lead, Instructions, Fuse (inside fuse box behind electrical outlet).



Vector R & D Inc. - 6824 19th St. #230 - University Place, WA 98466
877-883-7455 Fax 253-564-4927
WWW.VECTORUSA.NET